

Instruction for the processing of various body fluids in the
Cellspin Cyto-centrifuge

Specimen: Oral mucosa

Sampling: With Rovers Orcellex brush JC347

Fixation: The brush is laid into a plastic container with screw cap, prefilled
CELLSPIN COLLECTION FLUID

Transport: By mail

Cyto-centrifugation: The slide is inserted into the clip with the ECOseal and
the ECOfunnel.

Pipette 2 ml of the sample into the ECOfunnel and
centrifuge at 1000 rpm for 5 min.

The clip with the slide, the filter paper and the ECOfunnel must be hung into the
rotor head before pipetting.

Recommendation: The material should be processed completely.

Fixation: CELLSPIN COLLECTION FLUID

The air-dried preparations can be stored for special analyses, like e.g.
immunocytochemistry.

Staining: Papanicolaou staining